

Hydromer Inc.
35 Industrial Parkway
Somerville, NJ. 08876 USA

&
Biosearch Medical Products, Inc.
(a subsidiary of Hydromer, Inc.)
Tel: 908-722-5000

UNCOATED
0.341

Hydromer coated (avg. 3 cycles)
0.044

Hydromer coated (25 cycles)
0.043

NOTE: Primer: 2-TS-96
Hydromer: 3-TS-12E
Process: 6-TS-88

Friction test:
COF= (Pull Force/ Sled Wgt)

Sled Wgt.= 350 gm = 0.77 lb

Test method is per Biosearch:
STM-92023, Rev. E

Wet Friction Tests (PEEK Tubing)

(HYDROMER tested 08/15/2006) Drag Sled = 350gr, Hydrated for 45 sec @ 23C

